# HistoCore SPECTRA CV Mounting Media

## HistoCore SPECTRA CV Cover Glass

English.....	2
العربية (Arabic).....	6
简体中文 (Chinese Simplified).....	9
中國傳統的 (Chinese Traditional).....	12
Dansk (Danish).....	15
Nederlands (Dutch).....	19
Français (French – Canada).....	23
Français (French – France).....	27
Deutsch (German).....	31
Italiano (Italian).....	35
日本語 (Japanese).....	39
한국어 (Korean).....	43
Norsk (Norwegian).....	48
Polski (Polish).....	52
Português (Portuguese – Brazil).....	56
Português (Portuguese – Portugal).....	60
Română (Romanian).....	64
Русский (Russian).....	68
Slovenski (Slovenian).....	72
Español (Spanish – Central America).....	76
Español (Spanish – Spain).....	80
Svensk (Swedish).....	84
ไทย (Thai).....	88
Türk (Turkish).....	91
Tiếng Việt (Vietnamese).....	95

# HistoCore SPECTRA CV X1 Mounting Media, CV Cover Glass

**REF** 3801733/3800152

## HistoCore SPECTRA CV X1 Mounting Media and CV Cover Glass

The HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass when used together produces a microscope slide of patient specimens that are preserved and protected from deterioration and/or physical disruption. This system has been optimized for use on the HistoCore SPECTRA CV Coverslipper.

### Intended Use

#### Detection/Measurement

The HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass do not detect or measure an analyte or marker. SPECTRA CV X1 Mounting Media is used to permanently adhere the SPECTRA CV Cover Glass to microscope slides. SPECTRA CV X1 Mounting Media when dried provides a seal between the SPECTRA Cover Glass and the microscope slide such that stained specimens are preserved and protected from deterioration and/or physical disruption.

#### Product Function

The HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass function as aids to diagnosis. SPECTRA CV X1 Mounting Media is used to permanently adhere the SPECTRA CV Cover Glass to microscope slides and in so doing provides a means of protecting the stained specimens from deterioration and/or physical disruption. When used together, it provides a refractive index that facilitates visualization using light microscopy of the histologic and cytologic specimen to aid in diagnosis.

#### Specific Information Provided

The HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass are not intended for the detection, definition or differentiation of a specific disorder, condition or risk factor. SPECTRA CV X1 Mounting Media and SPECTRA CV Cover Glass are used as a means of protecting stained histologic or cytologic specimens from deterioration or physical disruption. The stained specimens when interpreted by a qualified pathologist may provide valuable data that is useful in the assessment of various physiological or pathological states.

#### Automation

The HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass are designed such that use of these devices is limited to the HistoCore SPECTRA CV Coverslipper.

#### Quality/Quantity

The HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass do not detect or measure an analyte or marker and is therefore neither qualitative, semiquantitative nor quantitative.

#### Specimen Type

The HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass can be used with any specimens mounted on a 25mm x 75mm x 1mm size glass microscope slide.

#### Testing Population

The HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass do not target a specific patient group. The SPECTRA CV X1 Mounting Media and SPECTRA CV Cover Glass are intended for use with any patient requiring histopathologic or cytologic evaluation of biopsy or resection tissue for the assessment of a suspected pathology or disease.

### In Vitro Diagnostic

The HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass are intended for *in vitro diagnostics* use only.

### Intended User

The HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass are intended for use by qualified laboratory personnel and/or designee.

### Test Principle

The HistoCore SPECTRA CV X1 Mounting Media, once the solvent is evaporated, leaves a solidified resinous base that seals the HistoCore SPECTRA CV Cover Glass onto the glass microscope slide for specimen preservation and visualization using light microscopy.

### Calibrators & Controls

The HistoCore CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass do not require the use of any calibrators or controls.

### Reagent Limitations

The HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass are designed for use only with the HistoCore SPECTRA CV Coverslipper. SPECTRA CV X1 Mounting Media should not be used on stains or patient specimens that are aqueous or not compatible with xylene or xylene substitutes.

### Products Provided

**HistoCore SPECTRA X1 Mounting Media** (REF 3801733, 300mL)

**HistoCore SPECTRA CV Cover Glass** (REF 3800152, 2,400 Cover Glass)

# HistoCore SPECTRA CV X1 Mounting Media, CV Cover Glass

**REF** 3801733/3800152

## Materials Not Included

The HistoCore SPECTRA X1 Mounting Media and HistoCore SPECTRA CV Cover Glass does not include microscope slides.

## Devices Required

The HistoCore SPECTRA X1 Mounting Media and HistoCore SPECTRA CV Cover Glass are designed for use with the HistoCore SPECTRA CV Coverslipper.

## Storage and Stability

Store reagents at room temperature (15-30°C). For HistoCore SPECTRA X1 Mounting Media please follow facility guidelines for storage of flammable products.

**CAUTION:** Do not use after the expiration date.

## In Use Stability

The HistoCore SPECTRA CV X1 Mounting Media has a 14 day on instrument shelf life. After 14 days, the mounting media should be removed from the SPECTRA CV Coverslipper and discarded in accordance with the applicable on-site lab guidelines.

**NOTE:** We recommend making a note of the use-date on the bottle label as the instrument does not alert end user of the on-board expiration.

## Sterility

The HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass are not sterile products.

## Warnings/Precautions

Unused cover glass should be disposed of in a container designated per facility guidelines. Unused mounting media should be disposed of per facility guidelines for hazardous material.

## Infectious Material Status

The HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass do not include any infectious material. However, specimens, before and after fixation, and all materials exposed to them, should be handled as if capable of transmitting infection and disposed of with proper precautions per facility guidelines.

## Special Facilities

The HistoCore SPECTRA CV X1 Mounting Media should be used per facility guidelines.

## Specimen Handling

Specimens intended for use with the HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass should be compatible with synthetic resins and xylene. The SPECTRA CV X1 Mounting Media and SPECTRA CV Cover Glass should not be used on specimens that require an aqueous mounting media and/or xylene substitute.

## Preparation for Use

### Placement of HistoCore SPECTRA CV Cover Glass on Instrument:

To place the HistoCore SPECTRA CV Cover Glass on the HistoCore SPECTRA CV Coverslipper, unpack the coverglass cartridge (Fig. 1-1) and remove the foam insert (Fig. 1-2) and the silica gel packet (Fig. 1-3). Open the hood of the HistoCore SPECTRA CV Coverslipper. Insert the coverglass cartridge into the designated slot on the instrument (Pick&Place module). Close the instrument hood. The coverglass cartridge data is then imported to the instrument and the Module Status screen on the instrument is updated. For additional information, please refer the HistoCore SPECTRA CV Coverslipper Instructions for Use (IFU).

Fig. 1

# HistoCore SPECTRA CV X1 Mounting Media, CV Cover Glass

**REF** 3801733/3800152

## Placement of the HistoCore SPECTRA CV X1 Mounting Media:

Take the HistoCore SPECTRA CV X1 Mounting Media bottle out of the packaging and remove the black cover cap (Fig. 2-1). Open the hood of the HistoCore SPECTRA CV Coverslipper, grasp the bottle sledge handle (top left of instrument) and move it all the way forward. Insert the mounting media bottle into the recess labeled L1 and make sure that the RFID chip (Fig. 2-4) is seated in the notch of the recess. Carefully insert the cannula with the label L1 into the opening of the mounting media until it clicks into place. This process involves puncturing the white protective membrane. Repeat the process for coverslipping line L2. Slide the bottle sledge back until you feel it click into place. Close the hood. The mounting media bottles that are inserted are detected by the instrument software and the Module Status screen of the instrument is updated.

**CAUTION:** Do not remove the white plastic seal (Fig. 2-2). It must remain on the bottle.

**CAUTION:** Puncturing the white protective membrane briefly requires an increase in force. Therefore, proceed carefully to avoid slipping from your hand.

**NOTE:** Before the insertion of a mounting media bottle in the instrument, the expiration date (imprint on the packaging and on the bottle label (Fig. 2-3)) is to be checked. If the expiration date has been reached or exceeded, the mounting media may no longer be used. Expired mounting media is to be disposed of in accordance with the applicable on-site lab guidelines.

**NOTE:** Opened mounting media has an on-board life of 14 days, within which it must be spent. We recommend making a note of the use-date on the bottle label as the instrument does not alert end user of the on board expiration.

**NOTE:** Do not dispose of the black cover cap. If a non-empty bottle of mounting medium is taken out of the instrument (e.g. in the event of transport), it can be closed-up again using the black plastic cap and retained. While doing so, the expiration date of the opened bottle must be observed.

**NOTE:** The software identifies the insertion of the new bottles and informs the user that an Extended Prime is necessary for both coverslip lines. Only afterwards is the instrument ready to start. Refer to the HistoCore SPECTRA CV Coverslipper for instruction on Extended Prime.

Fig. 2

## Readiness for Use

Once the HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass are ready to be used, the consumables status display (bottom right of screen) under the Process Status menu tab (first tab down on the left side of the screen) on the HistoCore SPECTRA CV Coverslipper will display the volume of each consumable available for use. When the volume of consumables is insufficient for slide processing, remove the consumables that are affected and follow the instructions above in the Preparation for Use for the respective consumable.

**NOTE:** If there is still cover glass present in the magazine, up to 30 slips of the remaining cover glass can be inserted in the new cover glass cartridge. These are added on to the new cover glass cartridge and displayed in the Module Status. Ensure that the cover glass is inserted correctly in the magazine. When the user is asked, "Did you transfer the remaining coverglass on top of the new cartridge in L1/L2?" select yes if cover glass was transferred to the new cartridge or select no if no transfer occurred.

## Quality Control

A routine control slide should be processed in a similar manner as the patient specimen to confirm the HistoCore SPECTRA CV Coverslipper is functioning properly prior to use.

## Expected Results

By following the instruction for use, a single HistoCore SPECTRA CV Cover Glass cartridge should provide 300 coverslipped specimen slides for visualization with light microscopy. One HistoCore SPECTRA CV X1 Mounting Media bottle should provide 2,100 coverslipped specimen slides for visualization with light microscopy.

# HistoCore

## SPECTRA

### CV X1 Mounting Media, CV Cover Glass

**REF** 3801733/3800152

#### Analytical Performance

The HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass are not used to detect a specific analyte or marker. The SPECTRA CV X1 Mounting Media is used to permanently adhere the SPECTRA CV Cover Glass to microscope slides. SPECTRA CV X1 Mounting Media when dried provides a seal between the SPECTRA Cover Glass and the microscope slide such that stained specimens are preserved and protected from deterioration and/or physical disruption. Analytical parameters such as analytical sensitivity, analytical specificity, trueness (bias), precision (repeatability and reproducibility), accuracy (resulting from trueness and precision), limits of detection and quantitation, measuring range, linearity, cut-off, including determination of appropriate criteria for specimen collection and handling and control of known relevant endogenous and exogenous interference, cross-reactions do not apply to the performance of the SPECTRA CV X1 Mounting Media and the SPECTRA CV Cover Glass.

#### Clinical Performance

The HistoCore SPECTRA CV X1 Mounting Media and HistoCore SPECTRA CV Cover Glass are not intended for use as a means of detecting a specific disease or pathological process or state. Clinical performance indices such as diagnostic sensitivity, diagnostic specificity, positive predictive value, negative predictive value, likelihood ratio as well as expected values in normal and affected populations do not apply to the use of the SPECTRA CV X1 Mounting Media and SPECTRA CV Cover Glass in a clinical setting.

#### Disposal

Unused and/or expired cover glass should be disposed of in a container designated per facility guidelines. Unused and/or expired mounting media should be disposed of per facility guidelines for hazardous material.

Leica Biosystems Richmond, Inc.  
5205 Route 12  
Richmond, IL 60071  
USA  
(1-844-534-2262)

[LeicaBiosystems.com](http://LeicaBiosystems.com)

CEpartner4U  
Esdoornlaan 13  
3951 DB Maarn  
The Netherlands  
cepartner4u.eu

Issue Date: 06/2021, Rev A • RM IFU-019

# HistoCore SPECTRA CV X1 Mounting Media, CV Cover Glass

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