HistoCore SPECTRA H&E Staining Systems

S1-800 & S2-800

English.....	2
العربية (Arabic).....	9
简体中文 (Chinese Simplified).....	16
中國傳統的 (Chinese Traditional).....	23
Dansk (Danish).....	30
Nederlands (Dutch).....	37
Français (French – Canada).....	45
Français (French – France).....	53
Deutsch (German).....	61
Italiano (Italian).....	69
日本語 (Japanese).....	77
한국어 (Korean).....	84
Norsk (Norwegian).....	91
Polski (Polish).....	98
Português (Portuguese – Brazil).....	106
Português (Portuguese – Portugal).....	114
Română (Romanian).....	122
Русский (Russian).....	130
Slovenski (Slovenian).....	140
Srpski (Serbian).....	147
Español (Spanish – Central America).....	155
Español (Spanish – Spain).....	163
Svensk (Swedish).....	171
ไทย (Thai).....	178
Türk (Turkish).....	188
Tiếng Việt (Vietnamese).....	194

HistoCore SPECTRA H&E Staining System

S1-800  **3801656/3801656CN**

S2-800  **3801657/3801657CN**

HistoCore SPECTRA H&E Staining System S1-800/S2-800

The HistoCore SPECTRA H&E Staining System S1-800/S2-800 contains 5 proprietary reagent components and staining protocols that when used together provide consistent diagnostic quality and ease of use. This system has been optimized for use on HistoCore SPECTRA ST stainers.

Intended Use

Detection/Measurement

The HistoCore SPECTRA H&E Staining System does not detect or measure an analyte or marker. HistoCore SPECTRA H&E Staining System is used to provide coloration and contrast to tissue sections mounted on microscopic slides from paraffin embedded specimens. The components of the stain system stain cell nuclei blue and color the connective tissue, cytoplasm, muscle and erythrocytes various shades of orange, pink and red.

Product Function

The coloration and contrast the HistoCore SPECTRA H&E Staining Systems provides to tissue sections allows visualization to microscopic anatomy. This visualization, when interpreted by a trained professional, is utilized alongside other information such as the patient's medical history, physical condition, as well as results from other medical testing to render a medical diagnosis.

Specific Information Provided

HistoCore SPECTRA H&E Staining System is not intended for the detection, definition or differentiation of a specific disorder, condition or risk factor. The staining demonstrated with use of this product, when used as intended, provides trained professionals information which may define the physiological or pathological state of the tissue specimen.

Automation

The HistoCore SPECTRA H&E Staining System is not automated but is intended for use on the HistoCore SPECTRA ST automated stainer.

Quality/Quantity

The HistoCore SPECTRA H&E Staining System is quantitative.

Specimen Type

The HistoCore SPECTRA H&E Staining System can be used with any paraffin embedded human or animal specimen.

Testing Population

The HistoCore SPECTRA H&E Staining System is intended for use with any patient requiring histopathologic evaluation of biopsy or resection tissue for the assessment of a suspected pathology or disease.

In Vitro Diagnostic

The HistoCore SPECTRA H&E Staining System is intended for *in vitro diagnostics* use only.

Intended User

The HistoCore SPECTRA H&E Staining System is intended for use by qualified laboratory personnel and/or designee of the laboratory.

Test Principle

The HistoCore SPECTRA H&E Staining System functions through the attraction of charged dyes to oppositely charged molecules within cells and tissues.

Calibrators & Controls

The HistoCore SPECTRA H&E Staining System does not require the use of any calibrators or controls. It is recommended that control slides containing tissue fixed and processed in a similar manner to the patient specimens should be included to ensure reagents are performing and functioning properly prior to staining the test specimens.

Reagent Limitations

The HistoCore SPECTRA H&E Staining System contains 5 proprietary reagent components. The reagents in the HistoCore SPECTRA H&E Staining System have been developed and optimized exclusively for use on the HistoCore SPECTRA ST Stainer. The HistoCore SPECTRA H&E Staining System is designed to be utilized with a tap water hook up to the HistoCore SPECTRA ST Stainer. The HistoCore SPECTRA H&E Staining System is developed and optimized for use with reagent grade alcohols and xylene.

Reagents Provided

HistoCore SPECTRA H&E Staining System S1-800

SPECTRA Hemalast S1-800 ( 3801585, 380mL)

SPECTRA Hematoxylin S1-800 ( 3801586, 380mL)

SPECTRA Differentiator S1-800 ( 3801587, 380mL)

SPECTRA Bluing Agent S1-800 ( 3801588, 380mL)

SPECTRA Eosin S1-800 ( 3801589, 380mL)

HistoCore SPECTRA H&E Staining System

S1-800 **REF** 3801656/3801656CN

S2-800 **REF** 3801657/3801657CN

HistoCore SPECTRA H&E Staining System S2-800

SPECTRA Hemalast S2-800 (REF 3801595, 380mL)
SPECTRA Hematoxylin S2-800 (REF 3801596, 380mL)
SPECTRA Differentiator S2-800 (REF 3801597, 380mL)
SPECTRA Bluing Agent S2-800 (REF 3801598, 380mL)
SPECTRA Eosin S2-800 (REF 3801599, 380mL)

Materials Not Included

The HistoCore SPECTRA H&E Staining System has been designed for tissue specimens placed on non-adhesive microscope slides. The HistoCore SPECTRA H&E Staining System requires the use of reagent grade alcohols and xylene.

Devices Required

The HistoCore SPECTRA H&E Staining System is developed and optimized exclusively for use with HistoCore SPECTRA ST Stainer.

Storage and Stability

Store reagents at room temperature (15-30°C). Please follow facility guidelines for storage of flammable products.

CAUTION: Do not use after the expiration date.

NOTE: Do NOT dispose of the staining system box. It is required for scanning the box label.

In Use Stability

The HistoCore SPECTRA H&E Staining System has a 7 day in use stability and a slide capacity of 800 slides. After 7 days on the HistoCore SPECTRA ST Stainer or after staining 800 slides, whichever occurs first, the reagents of the HistoCore SPECTRA H&E Staining System should be disposed in accordance with local governing regulations. When the reagents are not in use, the reagent vessels should be covered to reduce evaporation to maintain performance characteristics.

Sterility

The HistoCore SPECTRA H&E Staining System is not a sterile product.

Warnings/Precautions

A routine quality control slide(s) containing tissue fixed and processed in a similar manner to the test specimens should be performed prior to routine use to ensure reagents are performing as intended. Reagents should be disposed of per facility guidelines for hazardous material.

Infection Material Status

The HistoCore SPECTRA H&E Staining System does not include any infectious material. However, specimens, before and after fixation, and all materials exposed to them, should be handled as if capable of transmitting infection and disposed of with proper precautions per facility guidelines.

Special Facilities

The HistoCore H&E Staining System does not require any use of special facilities.

Specimen Handling

Specimens intended for use with the HistoCore SPECTRA H&E Staining System should be well fixed with neutral buffered formalin. Following processing and paraffin embedding, section tissue at a standard thickness of (2 – 5µm).

Preparation for Use

Settings for operational parameters of the HistoCore SPECTRA ST stainer and protocol schedules are provided to optimize the performance and consistency of the HistoCore SPECTRA H&E Staining System.

NOTE: Do NOT dispose of the staining system box. It is required for scanning the box label.

Initial Set-up of HistoCore SPECTRA H&E Staining System Protocol:

Initial set up must be conducted using the supervisor mode on the HistoCore SPECTRA ST Stainer. The HistoCore SPECTRA H&E Staining System S1-800/S2-800 protocols can be located under the “Programs” button on the left side of screen on the HistoCore SPECTRA ST Stainer. To open, press the “Programs” button on the touch screen. (note: Protocols and reagent names as displayed on the screen contain a lower case “s”.)

Assigning a Rack Handle Color:

- To assign a rack handle color to the HistoCore SPECTRA H&E Staining System S1-800 s/S2-800 s Protocol, tap the program in the program list to select it.
- Press the “Color” button located on the right side of the screen.
- The color display will open on the screen. Select the desired color by pressing that color on the screen. When completed, press the “Save” button located on the bottom right of the screen. This will save the color choice and close the window.

Oven Specifications and Staining Intensity:

HistoCore SPECTRA H&E Staining System

S1-800  3801656/3801656CN

S2-800  3801657/3801657CN

The HistoCore SPECTRA H&E Staining System S1-800 /S2-800 protocols are loaded with a default setting. These default settings include the use of the oven to dry the slides. To de-activate the use of the oven prior to staining, select the protocol under the “Programs” tab and press the “Edit” button on the right hand side of the screen. A new window will open displaying the oven, hematoxylin, and eosin settings. To de-activate the oven, click the box next to “Exclude Oven Step”. The hematoxylin and eosin settings are both defaulted to 4. To increase or decrease the hematoxylin staining intensity, either increase or decrease the number on the hematoxylin dial. To increase or decrease the eosin staining intensity, either increase or decrease the number on the eosin dial. Once changes have been made, press the “Save” button on the bottom right hand side of the window. The window will close and the changes will be saved.

NOTE: If the oven step is not being utilized on the instrument prior to staining, slides should be dried at 66 - 74°C for 15-22 minutes.

Defining a Bathlayout:

Single S1-800/S2-800 Staining System:

To use the HistoCore SPECTRA H&E Staining System S1-800/S2-800, a bathlayout must be generated. Follow the steps below to generate a bathlayout containing the HistoCore SPECTRA H&E Staining System S1-800/S2-800 protocol:

- Under the “Program” tab, press the button “Define Bathlayout” located at the bottom of the screen.
 - A new window will open. In this window, select the box under the “Use” column next to “SPECTRA H&E S1-800 s” or “SPECTRA H&E S2-800 s”.
- NOTE:** If additional protocols wish to be generated, they must be selected at this time.
- Once all protocols have been selected, press the button “Execute Bathlayout” at the bottom right hand of the screen.
- A message will appear containing information about the need to use fresh reagents. Press “Continue” to proceed, or “Cancel” to abort bathlayout execution.
- CAUTION:** Once the “Continue” button is pressed, any and all staining system(s) and reagents will be erased from the bathlayout, regardless of the current state of the staining system(s) or reagents. This will make any staining system(s) and reagents unusable.
- Once “Continue” has been pressed, a new window will open. This window will ask you to confirm the protocols selected to be adapted in the bathlayout. Press “Continue” to proceed.
 - A screen demonstrating the bathlayout will open. A message will appear at the top of screen stating “For validated HistoCore SPECTRA H&E program(s) the 95% Alcohol Dehyd 1 (95Dhy 1) must be duplicated using the function “Adapt Bathlayout.” To complete this step, press the “Adapt Bathlayout” button. An additional window will open on the screen. This window is called a “Clipboard” window. An identical copy of the selected station will appear on the “Clipboard”. Select the station in the “Clipboard” and then click on a free position in the bathlayout. The free station selected should be as close to the SPECTRA Eosin S1-800 s A/S2-800 s A (L Eos S1sA or L Eos S2sA) and 100% Alcohol Dhyd 1 H (100Dhy 1H) as possible. The duplicated station has now been moved from the clipboard to the bathlayout. Press the “Finish” button on the “Clipboard” and confirm the subsequent message with “Yes” in order to apply the change and close the clipboard. Both 95Dhy 1 stations will now display an equivalence symbol (∞) in the bathlayout.

Dual S1 or S2 Staining Systems:

To use dual HistoCore SPECTRA H&E Staining System S1-800/S2-800, a bathlayout must be generated. Follow the steps below to generate a bathlayout containing a HistoCore SPECTRA H&E Staining System S1-800/S2-800 protocol with two H&E staining lines:

- Under the “Program” tab, press the button “Define Bathlayout” located at the bottom of the screen.
 - A new window will open. In this window, select the box under the “Use” column next to the corresponding staining system protocol desired: “SPECTRA H&E S1-800 s A” and “SPECTRA H&E S1-800 s B” or “SPECTRA H&E S2-800 s A” and “SPECTRA H&E S2-800 s B”.
- NOTE:** If additional protocols wish to be generated, they must be selected at this time.
- Once all protocols have been selected, press the button “Execute Bathlayout” at the bottom right hand of the screen.
- A message will appear containing information about the need to use fresh reagents. Press “Continue” to proceed, or “Cancel” to abort bathlayout execution.
- CAUTION:** Once the “Continue” button is pressed, any and all staining system(s) and reagents will be erased from the bathlayout, regardless of the current state of the staining system(s) or reagents. This will make any staining system(s) and reagents unusable.
- Once “Continue” has been pressed, a new window will open. This window will ask you to confirm the protocols selected to be adapted in the bathlayout. After confirming that the SPECTRA H&E S1-800 s A, SPECTRA H&E S1-800 s B or SPECTRA H&E S2-800 s A, SPECTRA H&E S2-800 s B and any additional protocols to be generated, press “Continue” and proceed.

HistoCore SPECTRA H&E Staining System

S1-800 REF 3801656/3801656CN


S2-800 REF 3801657/3801657CN

- A screen demonstrating the bathlayout will open. A message will appear at the top of screen stating, “For validated HistoCore SPECTRA H&E program(s) the 95% Alcohol Dehyd 1 (95Dhy 1) must be duplicated using the function Adapt Bathlayout.” To complete this step, press the “Adapt Bathlayout” button at the bottom of the screen. An additional window will open on the screen (Clipboard). Select the 95Dhy 1 station in the bathlayout. Select the “Duplicate” button on the Clipboard. The duplicated station will be placed onto the “Clipboard”. Both 95Dhy 1 stations will display an equivalence symbol (∞).

NOTE: To operate dual HistoCore SPECTRA H&E Staining System S1-800/S2-800 at the highest efficiency, it is recommended to duplicate all ancillary reagent stations, and quadruple the 95Dhy 1 station. To duplicate all ancillary reagent stations, follow the steps listed above. A suggested bathlayout with dual ancillary reagents and dual staining lines is demonstrated in Image 1 below.

Image 1 – Example of Dual H&E Staining System Bathlayout

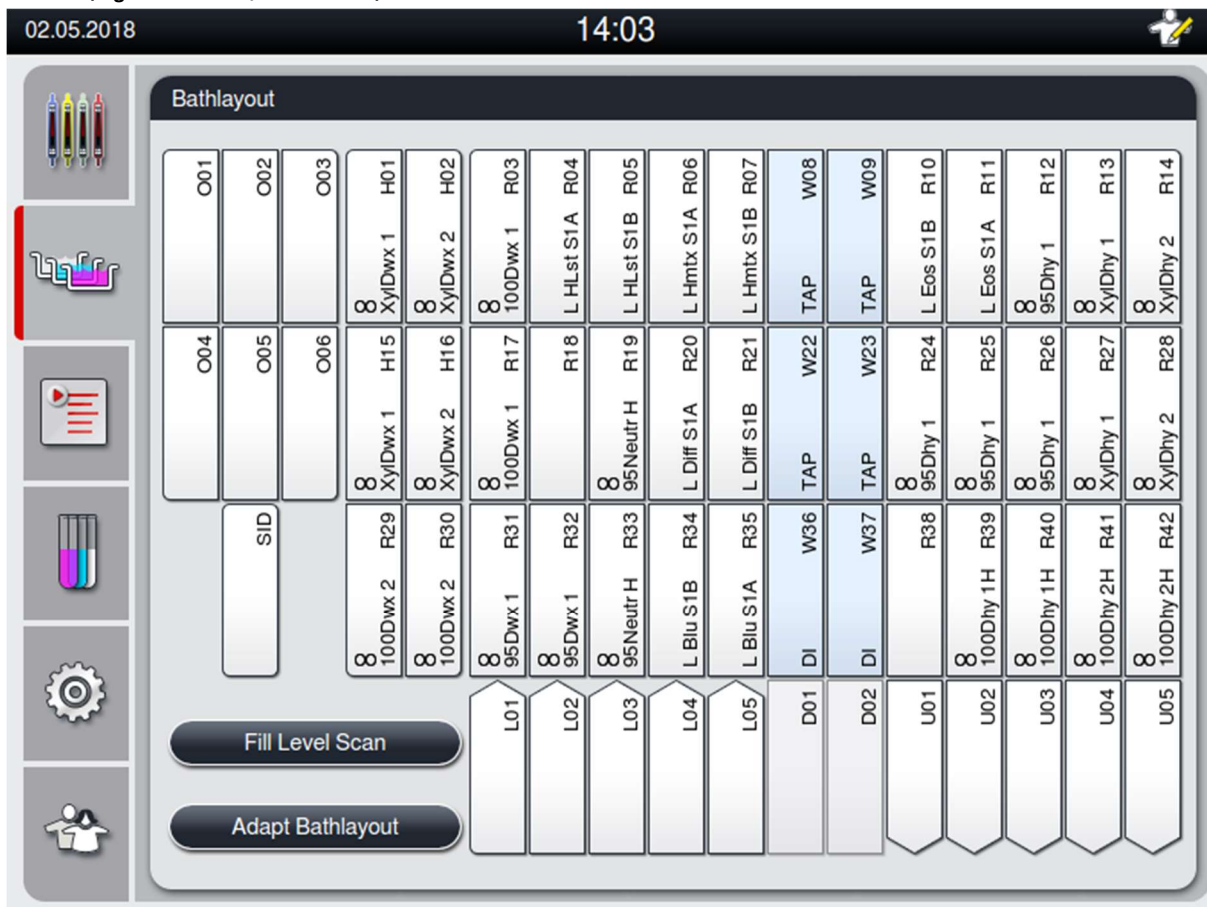NOTE: The dual system bathlayout will display a suffix to the staining reagents that corresponds to the staining system being utilized (e.g. S1sA/S1sB, S2sA/S2sB).

Filling Reagent Vessels

- All ancillary reagents need to be filled. To complete this, press the “Fill Reagents” button on the bottom of the screen. The “Fill Reagents” window will now open. All reagents that need to be filled will be outlined in orange. Fill the marked reagent vessels outside of the instrument with the corresponding reagents and set them back in the correct position in the instrument. Confirm filling and placement of the reagent vessels by pressing the respective station on the screen. The orange outline of that box will then disappear. Repeat this step until all stations marked in orange have been filled.

HistoCore SPECTRA

H&E Staining System

S1-800  3801656/3801656CN

S2-800  3801657/3801657CN

CAUTION: The arrangement of the reagent vessels is to be followed exactly. Any deviation can lead to undesired staining results.

- Once all ancillary reagents have been filled and accepted, the staining system is ready to be scanned. Select a staining reagent from the bathlayout screen. A message will pop up, prompting scanning of the staining system. Place the staining system container label up to the RFID sensor on the front, left side of the instrument. This will scan the staining system into the instrument software. After being prompted by a message, scan the first reagent bottle in the staining system by holding the reagent bottle label in front of the RFID sensor on the front, left side of the instrument. Reagent bottles may be scanned in any order. A new window will open displaying the bathlayout. The reagent that was scanned in will be outlined in orange on the bathlayout. Remove the reagent vessel from the instrument, open the reagent bottle, and pour all the reagent into the reagent vessel. Place the reagent vessel back into the respective station. Once this is completed, confirm filling and placement of the reagent vessel by pressing the respective station on the screen. Repeat this step until all 5 reagent bottles have been scanned and placed in their respective stations on the instrument. After all reagent vessels have been scanned in and filled, a window will open instructing to complete a "Fill Level Scan". Press "OK" to complete the fill level scan.

CAUTION: The user has 5 minutes to scan the reagents for the staining system packaging label and 5 minutes per reagent bottle. If reading in a reagent bottle or staining system label fails, the user has 2 additional attempts before the reagent labels become invalid.

NOTE: To scan a secondary staining system, select a staining reagent labeled S1sB/S2sB, and follow the directions above to complete scanning and filling of staining system and reagents.

NOTE: For additional instructions on defining bathlayouts, refer to the SPECTRA ST Instructions for Use.

Refilling Exhausted or Expired Staining Systems

- To replenish exhausted or expired staining systems, open the "Bathlayout" screen by pressing on the "Bathlayout" tab on the main menu. Press on any of the staining components. A window will open asking to scan the Leica kit label. Using the staining system box, hold the label of the staining system box up to the RFID sensor on the front, left side of the instrument. This will scan the new staining system into the instrument software. After being prompted by a message, scan the first reagent bottle in the staining system by holding the reagent bottle label in front of the RFID sensor on the front, left of the instrument. Reagent bottles may be scanned in any order. A new window will open displaying the bathlayout. The reagent that was scanned in will be outlined in orange on the bathlayout. Remove the reagent vessel from the instrument, open the reagent bottle, and pour all the reagent into the reagent vessel. Place the reagent vessel back into the respective station. Once this is completed, confirm filling and placement of the reagent vessel by pressing the respective station on the screen. Repeat this step until all 5 reagent bottles have been scanned and placed in their respective stations on the instrument. After all reagent vessels have been scanned in and filled, a window will open instructing to complete a "Fill Level Scan". Press "OK" to complete the fill level scan.

CAUTION: The user has 5 minutes to scan the reagents for the staining system packaging label and 5 minutes per reagent bottle. If reading of a reagent bottle or staining system label fails, the user has two additional attempts before the reagent labels become invalid.

Agitation Settings:

Agitation settings are predefined. Agitation will occur at a setting of 4. Refer to the HistoCore SPECTRA ST Instructions for Use.

Immersion Times:

Immersion times and instrument settings for HistoCore SPECTRA ST stainer are detailed in Table 2 below. Please follow instructions for operating HistoCore SPECTRA ST as detailed in the HistoCore SPECTRA ST Instructions for Use.

CAUTION: Modifications to the validated staining protocol may result in compromised staining quality as well as staining capacity.

CAUTION: All of the reagents in each HistoCore SPECTRA H&E Staining System S1-800/S2-800 must be used together to achieve the expected performance. Substitution of non-HistoCore SPECTRA H&E reagents or reagents from another HistoCore SPECTRA H&E Staining System will compromise performance.

CAUTION: Opening the lid of the HistoCore SPECTRA ST stainer during protocol execution may increase time in staining system reagent and impact the consistency in staining quality.

CAUTION: Reagent vessels should be covered when not in use to reduce evaporation.

CAUTION: If the HistoCore SPECTRA ST is not connected to the HistoCore SPECTRA CV, the slides will be transferred to the unload drawers. In this occurrence, ensure there is the appropriate reagent in the unload vessel, otherwise quality of staining may be compromised.

CAUTION: Any deviations from these instructions may alter expected results and are neither validated nor supported by Leica Biosystems.

HistoCore SPECTRA H&E Staining System

S1-800 **REF** 3801656/3801656CN

S2-800 **REF** 3801657/3801657CN

NOTE: Reagent vessels (including water rinse vessels) should be inspected at periodic intervals to ensure the integrity and cleanliness of the vessels. If cleaning solutions are used, rinse vessels with adequate water to remove any residual cleaning solutions prior to use. Clean the vessels in accordance with the instructions provided in the HistoCore SPECTRA ST Instructions for Use.

NOTE: The HistoCore SPECTRA ST Reagent Management System (RMS) tracks the number of slides that have utilized a specific station. However, for the ancillary reagents that have been rotated, the RMS will no longer capture the total number of slides that have utilized that specific ancillary reagent. For more information please refer to the SPECTRA ST Instructions for Use.

Table 1	Protocol Name: H&E Staining System S1-800/S2-800		Abbreviation: S1s or S2s		Agitation Speed: 4		Oven Temperature: 70°C	
Step	Reagent	Abbreviation	Time (mm:ss)	Tolerance	Process Class	Heated Vessel	Days Max	Slide Max
Default: Include Oven Step	Oven	Oven	15:00	50%	NA	NA	NA	NA
1	Xylene Dewax 1	XylDwx 1	2:00	100%	Dewaxing	No	5	800
2	Xylene Dewax 2	XylDwx 2	2:00	100%	Dewaxing	No	5	800
3	100% Alcohol Dewax 1	100Dwx 1	2:00	100%	Dewaxing	No	5	800
4	100% Alcohol Dewax 2	100Dwx 2	2:00	100%	Dewaxing	No	5	800
5	95% Alcohol Dewax 1	95Dwx 1	2:00	100%	Dewaxing	No	5	800
6	Tap Water	Tap Water	2:00	100%	NA	No	NA	NA
7	SPECTRA Hemalast S1-800s/ SPECTRA Hemalast S2-800s	L HLst S1s L HLst S2s	2:00	100%	Neutralizing	No	7	800
8	SPECTRA Hematoxylin S1-800s/ SPECTRA Hematoxylin S2-800s	L Hmtx S1s L Hmtx S2s	Refer to Hematoxylin Dial Settings	0%	Staining	No	7	800
9	Tap Water	Tap Water	2:00	100%	NA	No	NA	NA
10	SPECTRA Differentiator S1-800s/ SPECTRA Differentiator S2-800s	L Diff S1s L Diff S2s	S1-800: 1:30 S2-800: 1:00	0%	Differentiating	No	7	800
11	Tap Water	Tap Water	2:00	100%	NA	No	NA	NA
12	SPECTRA Bluing Agent S1-800s/ SPECTRA Bluing Agent S2-800s	L Blue S1s L Blue S2s	2:00	100%	Staining	No	7	800
13	Tap Water	Tap Water	2:00	100%	NA	No	NA	NA
14	95% Alcohol Neutralizing H	95Neutr H	2:00	100%	Neutralizing	No	5	800
15	SPECTRA Eosin S1-800s/ SPECTRA Eosin S2-800s	L Eos S1s L Eos S2s	Refer to Eosin Dial Settings	0%	Staining	No	7	800
16*	95% Alcohol Dehyd 1	95Dhy 1	2:00	0%	Dehydrating	No	5	400
17	100% Alcohol Dhyd 1 H	100Dhy 1H	2:00	0%	Dehydrating	No	5	800
18	100% Alcohol Dhyd 2 H	100Dhy 2H	2:00	0%	Dehydrating	No	5	800
19	Xylene Dehyd 1	XylDhy 1	2:00	100%	Dehydrating	No	5	800
20	Xylene Dehyd 2	XylDhy 2	2:00	100%	Dehydrating	No	5	800
21	Exit	NA	NA	NA	NA	NA	NA	NA

* Indicates a manually duplicated step.

Protocol Adjustments

The validated HistoCore SPECTRA H&E Staining System S1-800 /S2-800 protocols come pre-loaded on the HistoCore SPECTRA ST and will default to a dial setting of 4 for both the Hematoxylin and Eosin. Only the Hematoxylin and Eosin times may be adjusted when using these staining systems. All other protocol steps and times outlined in Table 1 are fixed. To adjust the staining intensity of the Hematoxylin and/or Eosin, use the Dial Settings in Table 2 below.

HistoCore SPECTRA H&E Staining System

S1-800 **REF** 3801656/3801656CN

S2-800 **REF** 3801657/3801657CN

Hematoxylin Dial Settings		Eosin Dial Settings	
Setting	Timing (mm:ss)	Setting	Timing (mm:ss)
Dial 1	1:00	Dial 1	0:30
Dial 2	2:00	Dial 2	1:00
Dial 3	3:00	Dial 3	1:30
Dial 4	4:00	Dial 4	2:00
Dial 5	5:00	Dial 5	2:30
Dial 6	6:00	Dial 6	3:00
Dial 7	7:00	Dial 7	3:30
Dial 8	8:00	Dial 8	4:00
Dial 9	9:00	Dial 9	4:30

Readiness for Use

Once the HistoCore SPECTRA H&E Staining System is placed onto the HistoCore SPECTRA ST Stainer, the system is ready to be used. If any reagent vessels that contain stain components or ancillary reagents are marked out with a red X on the bathlayout screen, this is an indication that the corresponding reagent vessel in the instrument is not ready to be used. Confirm that the reagent vessel is in the correct location in the instrument and the fill level of the reagent is sufficient in the reagent vessel. After confirming, repeat the fill level scan by selecting the Fill Level Scan button on the bathlayout screen. For further troubleshooting, refer to the HistoCore SPECTRA ST Instructions for Use.

Quality Control

A routine quality control slide(s) containing tissue fixed and processed in a similar manner to the test specimens should be performed prior to routine use to ensure reagents are performing as intended.

Expected Results

By following the instruction for use, a single HistoCore SPECTRA H&E Staining System S1-800/S2-800 should provide at least 800 stained slides of diagnostically consistent quality. The components of the stain system will stain cell nuclei blue and color the connective tissue, cytoplasm, muscle and erythrocytes various shades of orange, pink and red.

Analytical Performance

The HistoCore SPECTRA H&E Staining System is not used to detect a specific analyte or marker. The system is used to stain cell nuclei blue and connective tissue, cytoplasm, muscle and erythrocytes various shades of orange, pink and red. Analytical parameters such as analytical sensitivity, analytical specificity, trueness (bias), precision (repeatability and reproducibility), accuracy (resulting from trueness and precision), limits of detection and quantitation, measuring range, linearity, cut-off, including determination of appropriate criteria for specimen collection and handling and control of known relevant endogenous and exogenous interference, cross-reactions do not apply to the performance of this system.

Clinical Performance

The HistoCore SPECTRA H&E Staining System is not intended for use as a means of detecting a specific disease or pathological process or state. Clinical performance indices such as diagnostic sensitivity, diagnostic specificity, positive predictive value, negative predictive value, likelihood ratio as well as expected values in normal and affected populations do not apply to the use of the HistoCore SPECTRA H&E Staining System in a clinical setting.

Disposal

The HistoCore SPECTRA Staining System should be disposed in accordance with local governing regulations.

**HistoCore
SPECTRA
H&E Staining System**

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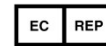
The screenshot shows the 'Bathlayout' interface with a grid of 48 stations. The stations are arranged in rows and columns, with labels such as O01-O06, R01-R42, W01-W03, and U01-U05. Each station contains a reagent code and a symbol (either a circle with an infinity symbol or a circle with a plus sign). The interface also includes a sidebar with icons for reagents, a graph, a tray, a gear, and a person, and buttons for 'Fill Level Scan' and 'Adapt Bathlayout'.

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The screenshot displays the 'Bathlayout' window of the HistoCore SPECTRA software. The interface includes a sidebar with icons for reagents, a graph, a test tube, a gear, and a person. The main area shows a grid of 48 stations, each with a unique ID and a reagent code. The stations are arranged in a grid with columns labeled O01-O06, R01-R42, W01-W03, and U01-U05. The reagent codes include XylDwx, 100Dwx, 95Dwx, 95Dhy, L HLst, L HLst, L Hmtx, L Hmtx, L Eos, L Eos, 95Dhy, XylDhy, XylDhy, and 100Dhy. The interface also features a 'Fill Level Scan' button and an 'Adapt Bathlayout' button at the bottom. The top of the screen shows the date '02.05.2018' and the time '14:03'.

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