English
Intended Use: To permanently preserve previously stained smears and routine microtome tissue sections 2 to 5 microns thick or stained cytological material for microscopic visualization. **Note:** These products are not suitable for any staining technique that is not able to tolerate alcohol.

Directions for Use:

- Wash stained slides in water or alcohol as necessary.
- Remove water from the tissue by replacing with several changes of graded alcohol.
- Replace the alcohol using several changes of a routine clearing agent. **Note:** Water and alcohol replacements may be achieved using an automatic staining machine. Times will vary according to the manufacturer. Refer to the manufacturer handbook if needed for guidance on timings.
- One (1) drop of mounting medium is then applied to the coverslip. The stained slide is then lowered onto the coverglass and the mounting medium is allowed to spread by capillary action. These steps may be achieved using an automatic coverslapping machine. Mechanisms will vary according to manufacturer (refer to the manufacturer handbook if needed).
- Allow slide to dry before examining the specimen microscopically.
- For removal of coverslip, soak the slides in xylene, toluene, or Sub X Clearing Agent, or alternatively, place the slides in an oven at 60°C for 3 to 4 hours to soften the mounting medium.

Additional Recommendations:

- Use only in a well ventilated area.
- Use only as much medium is needed to cover the coverslip.
- Allow slides to dry before moving to the microscope to avoid movement of the coverglass which may affect the viewing area.
- Allow time to dry thoroughly before final filing of slides to avoid them sticking together.
- The use of intense heat to dry the slides is not recommended. To expedite complete set-up of the mounting medium, slides may be placed in a spark proof convection oven on trays which allow for the passage and ventilation of air flow (45-50°C overnight is usually adequate). Slides should be allowed to come down to room temperature and harden before filing.
- Remove any excess mounting medium after positioning of the coverglass.
- Store stained slides away from light.
- Check a representative set of stored slides periodically for clarity.
- For optimum performance, ensure that the prepared material is of even thickness and that the mounting medium completely covers the material.
- Manipulate the coverglass as needed to ensure that there are no air bubbles prior to drying. Handle the mounting medium with minimal disturbances to avoid generation of air bubbles.
- Use only clean coverglass.
- When using an automatic coverslapping machine, clean the machine regularly in accordance with manufacturer instructions to avoid contamination of the mountant.

See Safety Data Sheet for additional information.

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