## BOND Dewax Solution

Catalog No: AR9222

Leica Biosystems Newcastle Ltd  
Balliol Business Park West  
Benton Lane  
Newcastle Upon Tyne NE12 8EW  
United Kingdom  
☎ +44 191 215 4242

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## Catalog No: AR9222

### Intended Use

This reagent is for *in vitro* diagnostic use.

BOND Dewax Solution is a ready to use dewaxing solution for removal of paraffin wax from formalin-fixed, paraffin-embedded tissue on the BOND automated system (includes Leica BOND-MAX system and Leica BOND-III system).

The clinical interpretation of any staining or its absence should be complemented by morphological studies and proper controls and should be evaluated within the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

### Summary and Explanation

Immunohistochemical techniques can be used to demonstrate the presence of antigens in tissue and cells (see "Using BOND Reagents" in your BOND user documentation).

The use of a dewaxing solution allows paraffin wax to be removed from tissue sections before rehydration and immunostaining on the BOND automated system (includes Leica BOND-MAX system and Leica BOND-III system).

### Reagents Provided

BOND Dewax Solution contains a solvent-based solution. Total volume = 1 L.

### Dilution and Mixing

BOND Dewax Solution is ready to use. Do not dilute this reagent.

### Materials Required But Not Provided

Refer to "Using BOND Reagents" in your BOND user documentation for a complete list of materials required for specimen treatment and immunohistochemical staining using the BOND system (includes Leica BOND-MAX system and Leica BOND-III system).

### Storage and Stability

BOND Dewax Solution should be stored in a well ventilated area away from direct sunlight, in accordance with local regulations. Store at 2–26 °C. Do not use after the expiration date indicated on the bottle label.

The signs indicating contamination and/or instability of BOND Dewax Solution are turbidity of the solution and presence of precipitate.

Storage conditions other than those specified above must be verified by the user<sup>1</sup>.

### Warnings and Precautions

- To obtain a copy of the Safety Data Sheet contact your local distributor or regional office of Leica Biosystems, or alternatively, visit the Leica Biosystems Web site, [www.LeicaBiosystems.com](http://www.LeicaBiosystems.com). The following information is taken from the product Safety Data Sheet:

Signal words: Danger.	H304: May be fatal if swallowed and enters airways.	P280: Wear protective gloves/protective clothing/eye protection/face protection.
GHS08: Health hazard.	EUH066: Repeated exposure may cause skin dryness or cracking.	P301+310: IF SWALLOWED: Immediately call a POISON CENTER or doctor.
		P331: Do NOT induce vomiting.
		P332+313: If skin irritation occurs: Get medical advice/attention.

- Specimens, before and after fixation, and all materials exposed to them, should be handled as if capable of transmitting infection and disposed of with proper precautions<sup>2</sup>. Never pipette reagents by mouth and avoid contacting the skin and mucous membranes with reagents or specimens. If reagents or specimens come in contact with sensitive areas, wash with copious amounts of water. Seek medical advice.
- Consult Federal, State or local regulations for disposal of any potentially toxic components.

### Instructions for Use

BOND Dewax Solution should be poured into the bulk container marked "Dewax Solution" located within the BOND Processing Module. This container can hold up to 2 L.

### Troubleshooting

Refer to reference 3 for remedial action.

Contact your local distributor or the regional office of Leica Biosystems to report unusual staining.

### Further Information

Further information on immunostaining with BOND reagents, under the headings Principle of the Procedure, Materials Required, Specimen Preparation, Quality Control, Assay Verification, Interpretation of Staining, Key to Symbols on Labels, and General Limitations can be found in "Using BOND Reagents" in your BOND user documentation.

### Bibliography

- Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
- Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
- Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.

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**Bibliographie**

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS), Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.

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Leica Biosystems Inc
Leica Biosystems Melbourne Pty Ltd