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BOND™ Ready-To-Use Primary Antibody Immunoglobulin D (DRN1C) Catalog No: PA0061

Intended Use

This reagent is for *in vitro* diagnostic use.

Immunoglobulin D (DRN1C) monoclonal antibody is intended to be used for the qualitative identification by light microscopy of Immunoglobulin D in formalin-fixed, paraffin-embedded tissue by immunohistochemical staining using the automated BOND system (includes Leica BOND-MAX system and Leica BOND-III system).

The clinical interpretation of any staining or its absence should be complemented by morphological studies and proper controls and should be evaluated within the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

Summary and Explanation

Immunohistochemical techniques can be used to demonstrate the presence of antigens in tissue and cells (see "Using BOND Reagents" in your BOND user documentation). Immunoglobulin D (DRN1C) primary antibody is a ready to use product that has been specifically optimized for use with BOND Polymer Refine Detection. The demonstration of human Immunoglobulin D is achieved by first allowing the binding of Immunoglobulin D (DRN1C) to the section, and then visualizing this binding using the reagents provided in the detection system. The use of these products, in combination with the automated BOND system (includes Leica BOND-MAX system and Leica BOND-III system), reduces the possibility of human error and inherent variability resulting from individual reagent dilution, manual pipetting and reagent application.

Reagents Provided

Immunoglobulin D (DRN1C) is a mouse anti-human monoclonal antibody produced as a tissue culture supernatant, and supplied in Tris buffered saline with carrier protein, containing 0.35 % ProClin™ 950 as a preservative.

Total volume = 7 mL.

Clone

DRN1C

Immunogen

Prokaryotic recombinant protein corresponding to 222 amino acids of the N terminus of the delta heavy chain constant region of the human immunoglobulin D molecule.

Specificity

Delta chain of human Immunoglobulin D.

Ig Class

IgG1

Total Protein Concentration

Approx 10 mg/mL.

Antibody Concentration

Greater than or equal to 0.19 mg/L as determined by ELISA.

Dilution and Mixing

Immunoglobulin D (DRN1C) primary antibody is optimally diluted for use on the BOND system (includes Leica BOND-MAX system and Leica BOND-III system). Reconstitution, mixing, dilution or titration of this reagent is not required.

Materials Required But Not Provided

Refer to "Using BOND Reagents" in your BOND user documentation for a complete list of materials required for specimen treatment and immunohistochemical staining using the BOND system (includes Leica BOND-MAX system and Leica BOND-III system).

Storage and Stability

Store at 2–8 °C. Do not use after the expiration date indicated on the container label.

The signs indicating contamination and/or instability of Immunoglobulin D (DRN1C) are: turbidity of the solution, odor development, and presence of precipitate.

Return to 2–8 °C immediately after use.

Storage conditions other than those specified above must be verified by the user¹.

Precautions

- This product is intended for *in vitro* diagnostic use.
- The concentration of ProClin™ 950 is 0.35 %. It contains the active ingredient 2-methyl-4-isothiazolin-3-one, and may cause irritation to the skin, eyes, mucous membranes and upper respiratory tract. Wear disposable gloves when handling reagents.
- To obtain a copy of the Material Safety Data Sheet contact your local distributor or regional office of Leica Biosystems, or alternatively, visit the Leica Biosystems' Web site, www.LeicaBiosystems.com

- Specimens, before and after fixation, and all materials exposed to them, should be handled as if capable of transmitting infection and disposed of with proper precautions². Never pipette reagents by mouth and avoid contacting the skin and mucous membranes with reagents or specimens. If reagents or specimens come in contact with sensitive areas, wash with copious amounts of water. Seek medical advice.
- Consult Federal, State or local regulations for disposal of any potentially toxic components.
- Minimize microbial contamination of reagents or an increase in non-specific staining may occur.
- Retrieval, incubation times or temperatures other than those specified may give erroneous results. Any such change must be validated by the user.

Instructions for Use

Immunoglobulin D (DRN1C) primary antibody was developed for use on the automated BOND system (includes Leica BOND-MAX system and Leica BOND-III system) in combination with BOND Polymer Refine Detection. The recommended staining protocol for Immunoglobulin D (DRN1C) primary antibody is IHC Protocol F. Heat induced epitope retrieval is recommended using BOND Epitope Retrieval Solution 2 for 40 minutes.

Results Expected

Normal Tissues

Clone DRN1C detected IgD expression in the cytoplasm and on the membrane of plasma cells, B cells of mantle zone and primary follicles and tissue infiltrating B lymphocytes, including tonsil, spleen, thymus and hypophysis. Staining was also seen in serum. (Total number of normal cases evaluated = 102).

Tumor Tissues

Clone DRN1C stained 2/10 diffuse large B cell lymphomas and 1/10 plasma cell myelomas. No staining was detected in Hodgkin's lymphomas (0/10), T cell lymphomas (0/8), follicular lymphomas (0/3) and Burkitt's lymphomas (0/3). Except for infiltrating lymphocytes no staining was observed in tumors of the thyroid (0/4), lung tumors (0/4), liver tumors (0/4), ovarian tumors (0/4), brain tumors (0/2), tumors of the esophagus (0/2), breast tumors (0/2), stomach tumors (0/2), soft tissue tumors (0/2), tumors of the tongue (0/2), metastatic tumors of unknown origin (0/2), kidney tumors (0/2) tumors of the cervix (0/2), testicular tumors (0/2), colon tumors (0/2), tumors of the rectum (0/2), skin tumors (0/2), a tumor of the larynx (0/1) or a tumor of the thymus (0/1). (Total number of abnormal cases evaluated = 88).

PA0061 is recommended for the identification of human immunoglobulin D protein in normal and neoplastic tissues.

Product Specific Limitations

Immunoglobulin D (DRN1C) has been optimized at Leica Biosystems for use with BOND Polymer Refine Detection and BOND ancillary reagents. Users who deviate from recommended test procedures must accept responsibility for interpretation of patient results under these circumstances. The protocol times may vary, due to variation in tissue fixation and the effectiveness of antigen enhancement, and must be determined empirically. Negative reagent controls should be used when optimizing retrieval conditions and protocol times.

Troubleshooting

Refer to reference 3 for remedial action.

Contact your local distributor or the regional office of Leica Biosystems to report unusual staining.

Further Information

Further information on immunostaining with BOND reagents, under the headings Principle of the Procedure, Materials Required, Specimen Preparation, Quality Control, Assay Verification, Interpretation of Staining, Key to Symbols on Labels, and General Limitations can be found in "Using BOND Reagents" in your BOND user documentation.

Bibliography

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Geisberger R, Lamers M and Achatz G. The riddle of the dual expression of IgM and IgD. Immunology. 2006; 118:429-437.
5. Preud'homme J, Petit I, Barra A et al. Structural and functional properties of membrane and secreted IgD. Molecular Immunology. 2000; 37:871-887.
6. Vladutiu A. Immunoglobulin D: Properties, Measurement, and Clinical Relevance. Clinical and diagnostic laboratory immunology. 2000; 7(2):131-140.

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Bibliographie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code : M9-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Geisberger R, Lamers M and Achatz G. The riddle of the dual expression of IgM and IgD. Immunology. 2006; 118:429-437.
5. Preud'homme J, Petit I, Barra A et al. Structural and functional properties of membrane and secreted IgD. Molecular Immunology. 2000; 37:871-887.
6. Vladuti A. Immunoglobulin D: Properties, Measurement, and Clinical Relevance. Clinical and diagnostic laboratory immunology. 2000; 7(2):131-140.

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Bibliografie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 28. February 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Geisberger R, Lamers M and Achatz G. The riddle of the dual expression of IgM and IgD. Immunology. 2006; 118:429-437.
5. Preud'homme J, Petit I, Barra A et al. Structural and functional properties of membrane and secreted IgD. Molecular Immunology. 2000; 37:871-887.
6. Vladutiu A. Immunoglobulin D: Properties, Measurement, and Clinical Relevance. Clinical and diagnostic laboratory immunology. 2000; 7(2):131-140.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Geisberger R, Lamers M and Achatz G. The riddle of the dual expression of IgM and IgD. Immunology. 2006; 118:429-437.
5. Preud'homme J, Petit I, Barra A et al. Structural and functional properties of membrane and secreted IgD. Molecular Immunology. 2000; 37:871-887.
6. Vladutiu A. Inmunoglobulin D: Properties, Measurement, and Clinical Relevance. Clinical and diagnostic laboratory immunology. 2000; 7(2):131-140.

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Bibliografia

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Geisberger R, Lamers M and Achatz G. The riddle of the dual expression of IgM and IgD. Immunology. 2006; 118:429-437.
5. Preud'homme J, Petit I, Barra A et al. Structural and functional properties of membrane and secreted IgD. Molecular Immunology. 2000; 37:871-887.
6. Vladutiú A. Imunoglobulina D: Properties, Measurement, and Clinical Relevance. Clinical and diagnostic laboratory immunology. 2000; 7(2):131-140.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code : M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Geisberger R, Lamers M and Achatz G. The riddle of the dual expression of IgM and IgD. Immunology. 2006; 118:429-437.
5. Preud'homme J, Petit I, Barra A et al. Structural and functional properties of membrane and secreted IgD. Molecular Immunology. 2000; 37:871-887.
6. Vladutiu A. Immunoglobulin D: Properties, Measurement, and Clinical Relevance. Clinical and diagnostic laboratory immunology. 2000; 7(2):131-140.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Geisberger R, Lamers M and Achatz G. The riddle of the dual expression of IgM and IgD. Immunology. 2006; 118:429-437.
5. Preud'homme J, Petit I, Barra A et al. Structural and functional properties of membrane and secreted IgD. Molecular Immunology. 2000; 37:871-887.
6. Vladutiú A. Immunoglobulin D: Properties, Measurement, and Clinical Relevance. Clinical and diagnostic laboratory immunology. 2000; 7(2):131-140.

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Bibliografie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Geisberger R, Lamers M and Achatz G. The riddle of the dual expression of IgM and IgD. Immunology. 2006; 118:429-437.
5. Preud'homme J, Petit I, Barra A et al. Structural and functional properties of membrane and secreted IgD. Molecular Immunology. 2000; 37:871-887.
6. Vladuțiu A. Immunoglobulin D: Properties, Measurement, and Clinical Relevance. Clinical and diagnostic laboratory immunology. 2000; 7(2):131-140.

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Leica Biosystems Newcastle Ltd
Leica Biosystems Inc
Pty Ltd