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### Catalog No: PA0379

#### Intended Use

This reagent is for *in vitro* diagnostic use.

p120 Catenin (EP66) monoclonal antibody is intended to be used for the qualitative identification by light microscopy of human p120 Catenin protein in formalin-fixed, paraffin-embedded tissue by immunohistochemical staining using the automated BOND system (includes Leica BOND-MAX system and Leica BOND-III system).

The clinical interpretation of any staining or its absence should be complemented by morphological studies and proper controls and should be evaluated within the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

#### Summary and Explanation

Immunohistochemical techniques can be used to demonstrate the presence of antigens in tissue and cells (see "Using BOND Reagents" in your BOND user documentation). p120 Catenin (EP66) primary antibody is a ready to use product that has been specifically optimized for use with BOND Polymer Refine Detection. The demonstration of p120 Catenin protein staining is achieved by first allowing the binding of p120 Catenin (EP66) to the section, and then visualizing this binding using the reagents provided in the detection system. The use of these products, in combination with the automated BOND system (includes Leica BOND-MAX system and Leica BOND-III system), reduces the possibility of human error and inherent variability resulting from individual reagent dilution, manual pipetting and reagent application.

#### Reagents Provided

p120 Catenin (EP66) is an affinity-purified rabbit anti-human monoclonal antibody, supplied in Tris buffered saline with carrier protein, containing 0.35 % ProClin™ 950 as a preservative.

Total volume = 7 mL.

#### Clone

EP66

N.B. This p120 Catenin antibody has been created by Epitomics Inc., using Epitomics' proprietary rabbit monoclonal antibody technology covered under Patent No.'s 5,675,063 and 7,402,409.

#### Immunogen

A synthetic peptide corresponding to residues in the human p120 protein.

#### Specificity

Human p120 protein.

#### Ig Class

Rabbit IgG.

#### Total Protein Concentration

Approx 10 mg/mL.

#### Antibody Concentration

Greater than or equal to 0.38 mg/L as determined by ELISA.

#### Dilution and Mixing

p120 Catenin (EP66) primary antibody is optimally diluted for use on the BOND system (includes Leica BOND-MAX system and Leica BOND-III system). Reconstitution, mixing, dilution or titration of this reagent is not required.

#### Materials Required But Not Provided

Refer to "Using BOND Reagents" in your BOND user documentation for a complete list of materials required for specimen treatment and immunohistochemical staining using the BOND system (includes Leica BOND-MAX system and Leica BOND-III system).

#### Storage and Stability

Store at 2–8 °C. Do not use after the expiration date indicated on the container label.

The signs indicating contamination and/or instability of p120 Catenin (EP66) are: turbidity of the solution, odor development, and presence of precipitate.

Return to 2–8 °C immediately after use.

Storage conditions other than those specified above must be verified by the user<sup>1</sup>.

#### Precautions

- This product is intended for *in vitro* diagnostic use.
- The concentration of ProClin™ 950 is 0.35 %. It contains the active ingredient 2-methyl-4-isothiazolin-3-one, and may cause irritation to the skin, eyes, mucous membranes and upper respiratory tract. Wear disposable gloves when handling reagents.
- To obtain a copy of the Material Safety Data Sheet contact your local distributor or regional office of Leica Biosystems, or alternatively, visit the Leica Biosystems' Web site, [www.LeicaBiosystems.com](http://www.LeicaBiosystems.com)

- Specimens, before and after fixation, and all materials exposed to them, should be handled as if capable of transmitting infection and disposed of with proper precautions<sup>2</sup>. Never pipette reagents by mouth and avoid contacting the skin and mucous membranes with reagents or specimens. If reagents or specimens come in contact with sensitive areas, wash with copious amounts of water. Seek medical advice.
- Consult Federal, State or local regulations for disposal of any potentially toxic components.
- Minimize microbial contamination of reagents or an increase in non-specific staining may occur.
- Retrieval, incubation times or temperatures other than those specified may give erroneous results. Any such change must be validated by the user.

### Instructions for Use

p120 Catenin (EP66) primary antibody was developed for use on the automated BOND system (includes Leica BOND-MAX system and Leica BOND-III system) in combination with BOND Polymer Refine Detection. The recommended staining protocol for p120 Catenin (EP66) primary antibody is IHC Protocol F. Heat induced epitope retrieval is recommended using BOND Epitope Retrieval Solution 1 for 20 minutes. Heat induced epitope retrieval is recommended using BOND Epitope Retrieval Solution 1 for 20 minutes.

### Results Expected

#### Normal Tissues

Clone EP66 detects p120 Catenin protein in the membrane and cytoplasm of a wide range of normal tissues. (Total number of normal tissues evaluated = 104).

#### Tumor Tissues

Clone EP66 stained 67/85 breast tumors (including 29/33 invasive ductal carcinomas, 20/32 invasive lobular carcinomas, and 18/20 fibroadenomas), 9/10 lymph nodes containing metastatic invasive ductal carcinoma, 4/4 adenocarcinomas of the gall bladder, 3/3 squamous cell carcinomas of the esophagus, 3/3 adenocarcinomas of the stomach, 3/3 carcinomas of the colon, 3/3 lung adenocarcinomas, 3/3 pancreatic adenocarcinomas, 3/3 transitional cell carcinomas of the bladder, 3/3 squamous cell carcinomas of the cervix, 1/3 renal clear cell carcinomas, 2/2 thyroid papillary carcinomas, 2/2 prostatic adenocarcinomas, and 1/1 melanoma. No staining was seen in astrocytomas (0/3). (Total number of abnormal cases evaluated = 131).

**p120 Catenin (EP66) is recommended for the assessment of p120 catenin protein expression in normal and neoplastic tissues.**

### Product Specific Limitations

p120 Catenin (EP66) has been optimized at Leica Biosystems for use with BOND Polymer Refine Detection and BOND ancillary reagents. Users who deviate from recommended test procedures must accept responsibility for interpretation of patient results under these circumstances. The protocol times may vary, due to variation in tissue fixation and the effectiveness of antigen enhancement, and must be determined empirically. Negative reagent controls should be used when optimizing retrieval conditions and protocol times.

### Troubleshooting

Refer to reference 3 for remedial action.

Contact your local distributor or the regional office of Leica Biosystems to report unusual staining.

### Further Information

Further information on immunostaining with BOND reagents, under the headings Principle of the Procedure, Materials Required, Specimen Preparation, Quality Control, Assay Verification, Interpretation of Staining, Key to Symbols on Labels, and General Limitations can be found in "Using BOND Reagents" in your BOND user documentation.

### Bibliography

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.

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## Bibliographie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code : M9-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.

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## Bibliografie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 28. February 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD und Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code : M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.

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Leica Biosystems Newcastle Ltd
Leica Biosystems Inc
Pty Ltd