

Kreatech™ FISH probes

Product Information Sheet


KBI-10609
CCND1 (11q13) Break



Kreatech Biotechnology B.V.
Vierweg 20
1032 LG Amsterdam
The Netherlands
www.LeicaBiosystems.com

PI-KBI-10609_D1.1

Published March 2015

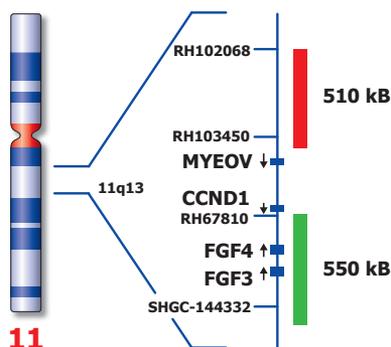

Not to scale

Kreatech™ CCND1 (11q13) Break FISH probe

Introduction: Overexpression of the Cyclin D1 gene caused by amplification or translocation is described for several types of cancer. A t(11;14) is the main characteristic aberration in mantle cell lymphoma. However, other translocation partners to CCND1 are also known (e.g. IGH at 2p12 and IGL at 22q11). A break or split assay is therefore best suited to detect rearrangements of the CCND1 gene region at 11q13.

Intended use: The **CCND1 (11q13) Break** FISH probe is optimized to detect translocations involving the CCND1 gene region at 11q13 in a dual-color, split assay on metaphase/interphase spreads.

The probe is recommended to be used in combination with one of the Kreatech Pretreatment kits providing necessary reagents to perform FISH on various sample types for optimal results. (see also www.LeicaBiosystems.com and look for Kits & reagents)

Critical region 1 (red): The proximal **CCND1** gene region probe is direct-labeled with PlatinumBright™550.
Critical region 2 (green): The distal **CCND1** gene region probe is direct-labeled with PlatinumBright™495.

Reagent: Kreatech probes are direct-labeled DNA probes provided in a ready-to-use format. Apply 10 µl of probe to a sample area of approximately 22 x 22 mm.

Please refer to the Instructions for Use for the entire Kreatech FISH protocol.

Kreatech FISH probes are REPEAT-FREE™ and therefore do not contain Cot-1 DNA. Hybridization efficiency is increased and background, due to unspecific binding, is highly reduced.

Interpretation: The **CCND1 (11q13) Break** FISH probe is designed as a dual-color break probe to detect translocations at 11q13. A break is defined when a red/green or yellow fusion signal (F) splits into separate red and green signals. Only red and green signals which are more than one signal diameter apart from each other are counted as a break. Co-localized red/green or yellow signals identify the normal chromosome(s) 11.

Signal patterns other than those described above may indicate variant translocations or other complex rearrangements. Investigators are advised to analyze metaphase cells for the interpretation of atypical signal patterns.

	Normal Signal Pattern	11q13 Break
Expected Signals	2F	1F1R1G

References: Vaandrager et al, 1996, Blood 88 (4); 1177-1182

Warning and precautions: In case of emergencies check SDS sheets for medical advice. SDS sheets may be obtained by either contacting Leica Technical Support or visiting www.LeicaBiosystems.com. DNA probes contain formamide which is a teratogen; do not inhale or allow skin contact. Wear gloves and a lab coat when handling DNA probes. All materials should be disposed of according to your institution's guidelines for hospital waste disposal.

Reagent Storage and Handling: Store at 2-8 °C. Reagents should not be used after the expiration date on the vial label.

TECHNICAL SUPPORT Technical support is available at www.LeicaBiosystems.com or +31 20 6919181 or via e-mail: kreatech-support@leicabiosystems.com.

CUSTOMER SERVICE Kreatech probes may be ordered through Leica Customer Service +31 20 6919181 or order via e-mail: purchase.orders@leica-microsystems.com.