

Kreatech™ FISH probes

Product Information Sheet

03C001N415

SE 3 (D3Z1)


Blue

50 µl

DANGER



FORMAMIDE

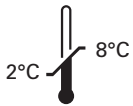


Kreatech Biotechnology B.V.
Vlierweg 20
1032 LG Amsterdam
The Netherlands
www.LeicaBiosystems.com

Analyte Specific Reagent

Analytical and performance characteristics are not established

PI-03C001N415_D3.1

Published August 2015

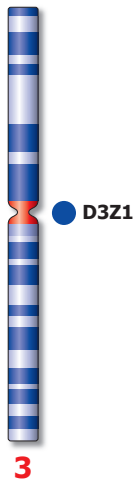

Not to scale

Kreatech™ SE 3 (D3Z1) FISH probe

- Introduction:** The **Satellite Enumeration (SE) 3 (D3Z1)** FISH probe is optimized to detect the chromosomes 3.
- Critical region (Blue):** The **SE 3 (D3Z1)** FISH probe is direct-labeled with PlatinumBright™415.
- Reagent:** The **SE 3 (D3Z1)** FISH probe is a direct-labeled DNA probe provided at ten times the concentration than the amount of DNA probe used in the QC test procedure.
- Kreatech FISH probes are REPEAT-FREE™ and therefore do not contain Cot-1 DNA. Hybridization efficiency is increased and background, due to unspecific binding, is highly reduced.**
- Patterns:** Two blue (2B) signals will identify the non-aberrant SE 3 loci.

	Signal Pattern
Expected Signals	2B

Warning and precautions: In case of emergencies check SDS sheets for medical advice. SDS sheets may be obtained by either contacting Leica Technical Support or visiting www.LeicaBiosystems.com. DNA probes contain formamide which is a teratogen; do not inhale or allow skin contact. Wear gloves and a lab coat when handling DNA probes. All materials should be disposed of according to your institution's guidelines for hospital waste disposal.

Reagent Storage and Handling:

Store at 2-8 °C. Reagents should not be used after the expiration date on the vial label.

TECHNICAL SUPPORT

Technical support is available at www.LeicaBiosystems.com/service-support/technical-support/ or toll free at 800-248-0123 or via e-mail: kreatech-support@leicabiosystems.com.

CUSTOMER SERVICE

Kreatech probes may be ordered through Leica Customer Service toll free at 800-248-0123 or order via e-mail: purchase.orders@leica-microsystems.com.