

Kreatech™ FISH probes Product Information Sheet

09C001J550

SE 9 (D9Z4)


Red

167 µl

DANGER



FORMAMIDE

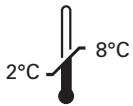


Kreatech Biotechnology B.V.
Vlierweg 20
1032 LG Amsterdam
The Netherlands
www.LeicaBiosystems.com

Analyte Specific Reagent

Analytical and performance characteristics are not established

PI-09C001J550_D1.1

Published Sept 2015

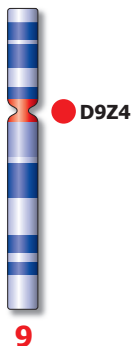

Not to scale

Kreatech™ SE 9 (D9Z4) FISH probe

- Introduction:** The **Satellite Enumeration (SE) 9 (D9Z4)** FISH probe is optimized to detect the chromosomes 9.
- Critical region 1 (red):** The **SE 9 (D9Z4)** FISH probe is direct-labeled with PlatinumBright™550
- Reagent:** The **SE 9 (D9Z4)** FISH probe is a direct-labeled DNA probe provided at three times the concentration than the amount of DNA probe used in the QC test procedure.
- Kreatech FISH probes are REPEAT-FREE™ and therefore do not contain Cot-1 DNA. Hybridization efficiency is increased and background, due to unspecific binding, is highly reduced.**
- Patterns:** Two red (2R) signals will identify the non-aberrant SE 9 loci.

| | Signal Pattern |
|------------------|----------------|
| Expected Signals | 2R |

Warning and precautions: In case of emergencies check SDS sheets for medical advice. SDS sheets may be obtained by either contacting Leica Technical Support or visiting www.LeicaBiosystems.com. DNA probes contain formamide which is a teratogen; do not inhale or allow skin contact. Wear gloves and a lab coat when handling DNA probes. All materials should be disposed of according to your institution's guidelines for hospital waste disposal.

Reagent Storage and Handling:

Store at 2-8 °C. Reagents should not be used after the expiration date on the vial label.

TECHNICAL SUPPORT

Technical support is available at www.LeicaBiosystems.com/service-support/technical-support/ or toll free at 800-248-0123 or via e-mail: kreatech-support@leicabiosystems.com.

CUSTOMER SERVICE

Kreatech probes may be ordered through Leica Customer Service toll free at 800-248-0123 or order via e-mail: purchase.orders@leica-microsystems.com.