

Kreatech™ FISH probes

Product Information Sheet

40V033V495

PapV-33, Flu labeled


Green

1 ml

DANGER



FORMAMIDE

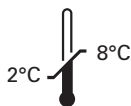

Kreatech Biotechnology B.V.
Vlierweg 20
1032 LG Amsterdam
The Netherlands
www.LeicaBiosystems.com

Analyte Specific Reagent

Analytical and performance characteristics are not established

40V033V495

Kreatech™ PapV-33, Flu labeled ISH probe

Introduction: The **PapV-33, Flu labeled** ISH probe is optimized to detect presence of HPV-33 DNA inserts in human tissue. The PapV-33, Flu labeled ISH probe is based on fluorescein labeling of the entire genomic sequence of the human Papilloma Virus type 33. Potential cross hybridization should be considered between homologous types when interpreting staining results.

Critical region (green): The **PapV-33, Flu labeled** ISH probe is direct-labeled with PlatinumBright™495. Probes labeled with PlatinumBright™495 can typically be detected using anti-Fluorescein antibodies.

Reagent: The **PapV-33, Flu labeled** ISH probe is a direct-labeled DNA probe provided at ten times the concentration than the amount of DNA probe used in the QC test procedure.

Kreatech probes are REPEAT-FREE™ and therefore do not contain Cot-1 DNA. Hybridization efficiency is increased and background, due to unspecific binding, is highly reduced.

Warning and precautions: In case of emergencies check SDS sheets for medical advice. SDS sheets may be obtained by either contacting Leica Technical Support or visiting www.LeicaBiosystems.com. DNA probes contain formamide which is a teratogen; do not inhale or allow skin contact. Wear gloves and a lab coat when handling DNA probes. All materials should be disposed of according to your institution's guidelines for hospital waste disposal.

Reagent Storage and Handling:

Store at 2-8 °C. Reagents should not be used after the expiration date on the vial label.

TECHNICAL SUPPORT

Technical support is available at www.LeicaBiosystems.com/service-support/technical-support/ or toll free at 800-248-0123 or via e-mail: kreatech-support@leicabiosystems.com.

CUSTOMER SERVICE

Kreatech probes may be ordered through Leica Customer Service toll free at 800-248-0123 or order via e-mail: purchase.orders@leica-microsystems.com.