

Novocastra Predilute EBER Probe

Product Code: ISH5687-A

Analyte Specific Reagent

Intended Use This reagent is an analyte specific reagent. Analytical and performance characteristics are not established.

Reagent Provided EBER probe is a fluorescein-conjugated oligonucleotide probe supplied in hybridization solution.
Total volume = 7 mL.

Specificity Latent EBV infection

Warnings and Precautions

EBER PROBE

Contains Formamide (<50%)
and Dextran Sulphate (<30%).
GHS07: Exclamation mark.
GHS08: Health hazard.
Signal words: Danger.

H315: Causes skin irritation.
H319: Causes serious eye irritation.
H360D: May damage the unborn child.

P201: Obtain special instructions before use.
P202: Do not handle until all safety precautions have been read and understood.
P260: Do not breathe dust/fumes/gas/mist/vapours/spray.
P281: Use personal protective equipment as required.
P308 + P313: IF exposed or concerned: Get medical advice/attention.
P314: Get medical advice/attention if you feel unwell.
P264: Wash thoroughly after handling.
P280: Wear protective gloves/protective clothing/eye protection/face protection.
P305 + P351 + P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P337 + P313: If eye irritation persists: Get medical advice/attention.
P302 + P352: IF ON SKIN: Wash with plenty of soap and water.
P332 + P313: If skin irritation occurs: Get medical advice/attention.
P362: Take off contaminated clothing and wash before reuse.
Restricted to professional users.

For additional information on toxicity please refer to the Material Safety Data Sheet (MSDS), which is available upon request, alternatively visit www.LeicaBiosystems.com

Statement of Quality

Each lot of reagent has been quality controlled by in situ hybridization staining.

Storage and Stability

Store at 2–8 °C in the dark. The product is stable under these conditions up to the expiry date indicated on the vial label.



General References

- Chang KL, Chen YY, Shibata D, Weiss LM. Description of an in situ hybridization methodology for detection of Epstein-Barr virus RNA in paraffin-embedded tissues, with a survey of normal and neoplastic tissues. *Diagn Mol Pathol*. 1992;1(4):246–255.
- Ambinder RF, Mann RB. Epstein-Barr-encoded RNA in situ hybridization: diagnostic applications. *Hum Pathol*. 1994; 25(6):602–605.
- Joncas J, Boucher J, Granger-Julien M, Filion C. Epstein-Barr virus infection in the neonatal period and in childhood. *CMAJ*. 1974;110:33–37.
- Niedermaier JC, McCollum RW, Henle G, Henle W. Infectious mononucleosis: clinical manifestations in relation to EB virus antibodies. *JAMA*. 1968;203:205–209.
- Sixbey JW, Vesterinen EH, Nedrud JG, Raab-Traub N, Walton LA, Pagano JS. Replication of Epstein-Barr virus in human epithelial cells infected in vitro. *Nature* 1983;306:480–483.
- Fingerroth JD, Weis JJ, Tedder TF, Strominger JL, Biro PA, Fearson DT. Epstein-Barr virus receptor of human B lymphocytes in the C3d receptor CR2. *Proc Natl Acad Sci U S A* 1984;81:4510–4514.
- Weiss LM, Movahed LA, Warnke RA, Sklar J. Detection of Epstein Barr viral genomes in Reed-Sternberg cells of Hodgkin's Disease. *NEJM*. 1989;302(8):502–506.
- Hummel M, Agnostopolous I, Korbjuhn P, Stein H. Epstein Barr virus in B cell non-hodgkins lymphoma. *J Pathol*. 1995;175(3):263–271.
- Brousset P, Bulet V, Chittal S, Selves J, Delsol G. Comparison of in situ hybridization using different isotopic probes for detection of nasopharyngeal carcinoma and immunohistochemical correlation with anti-latent membrane protein antibody. *Lab Invest*. 1992;67(4):457–464.
- Oudejans JJ, Jiwa M, van den Brule AJ, Grässer FA, Horstman A, Vos W, Kluin PM, van der Valk P, Walbloomers JM, Meijer CJ. Detection of heterogeneous Epstein-Barr virus gene expression patterns within individual post-transplant lymphoproliferative disorders. *Am J Pathol*. 1995;147(4):923–933.
- Luo B, Wang Y, Xiao-Feng W, Liang H, Yan L, Huang B, Zhao P. Expression of Epstein-Barr virus genes in EBV-associated carcinomas. *World J Gastroenterol*. 2005;11(5):629–633.
- Teramoto N, Sarker AB, Tonoyama Y, Yoshino T, Hyashi K, Takahashi K, Akagi T. Epstein-Barr virus infection in neoplastic and nonneoplastic cells of lymphoid malignancies. *Cancer*. 1996;77(11):2339–2347.
- Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
- Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
- Wilkinson DG. The theory and practice of in situ hybridization. In: Wilkinson DG. (ed.) *In Situ Hybridization A practical approach*. 2nd Edition. New York: Oxford University Press, 1998, pp.18–20.

Date of Issue

02 March 2015