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BOND™ Ready-To-Use Primary Antibody Cytokeratin 17 (E3) Catalog No: PA0114

Intended Use

This reagent is for in vitro diagnostic use.

Cytokeratin 17 (E3) monoclonal antibody is intended to be used for the qualitative identification by light microscopy human cytokeratin 17 in formalin-fixed, paraffin-embedded tissue by immunohistochemical staining using an automated BOND system (includes Leica BOND-MAX system and Leica BOND-III system).

The clinical interpretation of any staining or its absence should be complemented by morphological studies and proper controls and should be evaluated within the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

Summary and Explanation

Immunohistochemical techniques can be used to demonstrate the presence of antigens in tissue and cells (see "Using BOND Reagents" in your BOND user documentation). Cytokeratin 17 (E3) primary antibody is a ready to use product that has been specifically optimized for use with BOND Polymer Refine Detection. The demonstration of human cytokeratin 17 is achieved by first, allowing the binding of Cytokeratin 17 (E3) to the section, and then visualizing this binding using the reagents provided in the detection system. The use of these products, in combination with an automated BOND system, reduces the possibility of human error and inherent variability resulting from individual reagent dilution, manual pipetting and reagent application.

Reagents Provided

Cytokeratin 17 (E3) is a mouse anti-human monoclonal antibody produced as a tissue culture supernatant, and supplied in Tris buffered saline with carrier protein, containing 0.35% ProClin™ 950 as a preservative.

Total volume = 7 mL.

Clone

E3.

Immunogen

Keratin preparation from rat enterocytes.

Specificity

Human cytokeratin 17 intermediate filament protein.

Subclass

IgG2b.

Total Protein Concentration

Approx 10 mg/mL.

Antibody Concentration

Greater than or equal to 0.2 mg/L as determined by ELISA.

Dilution and Mixing

Cytokeratin 17 (E3) primary antibody is optimally diluted for use on a BOND system. Reconstitution, mixing, dilution or titration of this reagent is not required.

Materials Required But Not Provided

Refer to "Using BOND Reagents" in your BOND user documentation for a complete list of materials required for specimen treatment and immunohistochemical staining using a BOND system.

Storage and Stability

Store at 2–8 °C. Do not use after the expiration date indicated on the container label.

The signs indicating contamination and/or instability of Cytokeratin 17 (E3) are: turbidity of the solution, odor development, and presence of precipitate.

Return to 2–8 °C immediately after use.

Storage conditions other than those specified above must be verified by the user¹.

Precautions

- This product is intended for *in vitro* diagnostic use.
- The concentration of ProClin™ 950 is 0.35%. It contains the active ingredient 2-methyl-4-isothiazolin-3-one, and may cause irritation to the skin, eyes, mucous membranes and upper respiratory tract. Wear disposable gloves when handling reagents.
- To obtain a copy of the Material Safety Data Sheet contact your local distributor or regional office of Leica Biosystems, or alternatively, visit the Leica Biosystems' web site, www.LeicaBiosystems.com.

- Specimens, before and after fixation, and all materials exposed to them, should be handled as if capable of transmitting infection and disposed of with proper precautions². Never pipette reagents by mouth and avoid contacting the skin and mucous membranes with reagents or specimens. If reagents or specimens come in contact with sensitive areas, wash with copious amounts of water. Seek medical advice.
- Consult Federal, State or local regulations for disposal of any potentially toxic components.
- Minimize microbial contamination of reagents or an increase in non-specific staining may occur.
- Retrieval, incubation times or temperatures other than those specified may give erroneous results. Any such change must be validated by the user.

Instructions for Use

Cytokeratin 17 (E3) primary antibody was developed for use on an automated BOND system in combination with BOND Polymer Refine Detection. The recommended staining protocol for Cytokeratin 17 (E3) primary antibody is IHC Protocol F. Heat induced epitope retrieval is recommended using BOND Epitope Retrieval Solution 1 for 20 minutes.

Results Expected

Normal Tissues

Clone E3 stained basal epithelial cells of larynx and prostate, myoepithelial cells of salivary gland, myoepithelial cells, sebaceous glands and hair follicles of skin, and Hassall's corpuscles and epithelial cells of thymus. (Total number of cases stained = 96).

Tumor Tissues

Clone E3 detected squamous cell carcinomas of esophagus (2/2), larynx (1/1), tongue (2/2), lung (1/1), cervix (2/2), skin (2/2), and cholangiocarcinoma of liver (1/1). No staining was seen in a variety of other carcinomas (17/17), adenomas (4/4), adenocarcinomas (6/6), germ cell tumors (3/3), anaplastic astrocytoma of brain (1/1), choroid plexus papilloma of brain (1/1) and atypical carcinoid of thymus (1/1). (Total number of cases stained = 44).

Cytokeratin 17 (E3) is recommended for the detection of human cytokeratin intermediate filament protein (46 kD) identified as cytokeratin 17 in normal and neoplastic tissue and may be useful in a panel of antibodies for the diagnosis of squamous cell carcinoma.

Product Specific Limitations

Cytokeratin 17 (E3) has been optimized at Leica Biosystems for use with BOND Polymer Refine Detection and BOND ancillary reagents. Users who deviate from recommended test procedures must accept responsibility for interpretation of patient results under these circumstances. The protocol times may vary, due to variation in tissue fixation and the effectiveness of antigen enhancement, and must be determined empirically. Negative reagent controls should be used when optimizing retrieval conditions and protocol times.

Troubleshooting

Refer to reference 3 for remedial action.

Contact your local distributor or the regional office of Leica Biosystems to report unusual staining.

Further Information

Further information on immunostaining with BOND reagents, under the headings Principle of the Procedure, Materials Required, Specimen Preparation, Quality Control, Assay Verification, Interpretation of Staining, Key to Symbols on Labels, and General Limitations can be found in "Using BOND Reagents" in your BOND user documentation.

Bibliography

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin™ 950 is a trademark of Supelco, a part of Sigma-Aldrich Corporation.

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Bibliographie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin™ 950 est une marque de commerce de Supelco, membre du groupe Sigma-Aldrich Corporation.

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Bibliografia

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
 2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
 3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
- ProCin™ 950 è un marchio di fabbrica di Supelco, società del gruppo Sigma-Aldrich.

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Bibliografie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin™ 950 ist eine Marke von Supelco, einem Unternehmen der Sigma-Aldrich Corporation.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin™ 950 es una marca registrada de Supelco, parte de Sigma-Aldrich Corporation.

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Bibliografia

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin™ 950 é uma marca comercial de Supelco, parte da Sigma-Aldrich Corporation.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. Το ProClin™ 950 είναι εμπορικό σήμα της Supelco, τμήμα της Sigma-Aldrich Corporation.

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Bibliografi

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin™ 950 er et varemærke tilhørende Supelco, en del af Sigma-Aldrich Corporation.

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Bibliografi

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin™ 950 er et varemerke for Supelco, en del av Sigma-Aldrich Corporation.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin™ 950, Sigma-Aldrich Corporation'a bağlı Supelco'nun ticari markasıdır.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 1763 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin™ 950 е търговска марка на Supelco, част от Sigma-Aldrich Corporation.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
 2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
 3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
- A ProCln™ 950 a Sigma-Aldrich Corporation részét képező Supelco védjegye.

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Bibliografie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
 2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
 3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
- ProClin™ 950 este o marcă comercială a Supelco, care face parte din Sigma-Aldrich Corporation.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin™ 950 — торговая марка компании Supelco, входящей в состав корпорации Sigma-Aldrich.

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Bibliografia

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin™ 950 jest znakiem handlowym firmy Supelco, będącej częścią Sigma-Aldrich Corporation.

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Literatura

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin™ 950 je blagovna znamka družbe Supelco, ki je del družbe Sigma-Aldrich Corporation.

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Literatura

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin™ 950 je ochranná známka společnosti Supelco, která je součástí Sigma-Aldrich Corporation.

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Literatúra

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin™ 950 je ochranná známka spoločnosti Supelco, súčasťou spoločnosti Sigma-Aldrich Corporation.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
 2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
 3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York, 1996.
- يتمل ProCin™ 950 علامة تجارية لشركة Supelco، وهي جزء من مؤسسة Sigma-Aldrich Corporation.

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Leica Biosystems Newcastle Ltd
Leica Biosystems Inc
Pty Ltd