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# BOND™ Ready-To-Use Primary Antibody Calretinin (CAL6) Catalog No: PA0346

## Intended Use

This reagent is for *in vitro* diagnostic use.

Calretinin (CAL6) monoclonal antibody is intended to be used for the qualitative identification by light microscopy of human calretinin molecule in formalin-fixed, paraffin-embedded tissue by immunohistochemical staining using the automated BOND system (includes Leica BOND-MAX system and Leica BOND-III system).

The clinical interpretation of any staining or its absence should be complemented by morphological studies and proper controls and should be evaluated within the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

## Summary and Explanation

Immunohistochemical techniques can be used to demonstrate the presence of antigens in tissue and cells (see "Using BOND Reagents" in your BOND user documentation). Calretinin (CAL6) primary antibody is a ready to use product that has been specifically optimized for use with BOND Polymer Refine Detection. The demonstration of human calretinin molecule is achieved by first allowing the binding of Calretinin (CAL6) to the section, and then visualizing this binding using the reagents provided in the detection system, and then visualizing this binding using the reagents provided in the detection system. The use of these products, in combination with the automated BOND system (includes Leica BOND-MAX system and Leica BOND-III system), reduces the possibility of human error and inherent variability resulting from individual reagent dilution, manual pipetting and reagent application.

## Reagents Provided

Calretinin (CAL6) is a mouse anti-human monoclonal antibody produced as a tissue culture supernatant, and supplied in Tris buffered saline with carrier protein, containing 0.35 % ProClin™ 950 as a preservative.

Total volume = 7 mL.

## Clone

CAL6

## Immunogen

Prokaryotic recombinant protein corresponding to the entire human calretinin molecule.

## Specificity

Human calretinin molecule.

## Ig Class

IgG2a

## Total Protein Concentration

Approx 10 mg/mL.

## Antibody Concentration

Greater than or equal to 0.6 mg/L as determined by ELISA.

## Dilution and Mixing

Calretinin (CAL6) primary antibody is optimally diluted for use on the BOND system (includes Leica BOND-MAX system and Leica BOND-III system). Reconstitution, mixing, dilution or titration of this reagent is not required.

## Materials Required But Not Provided

Refer to "Using BOND Reagents" in your BOND user documentation for a complete list of materials required for specimen treatment and immunohistochemical staining using the BOND system (includes Leica BOND-MAX system and Leica BOND-III system).

## Storage and Stability

Store at 2–8 °C. Do not use after the expiration date indicated on the container label.

The signs indicating contamination and/or instability of Calretinin (CAL6) are: turbidity of the solution, odor development, and presence of precipitate.

Return to 2–8 °C immediately after use.

Storage conditions other than those specified above must be verified by the user<sup>1</sup>.

## Precautions

- This product is intended for *in vitro* diagnostic use.
- The concentration of ProClin™ 950 is 0.35 %. It contains the active ingredient 2-methyl-4-isothiazolin-3-one, and may cause irritation to the skin, eyes, mucous membranes and upper respiratory tract. Wear disposable gloves when handling reagents.
- To obtain a copy of the Material Safety Data Sheet contact your local distributor or regional office of Leica Biosystems, or alternatively, visit the Leica Biosystems' Web site, [www.LeicaBiosystems.com](http://www.LeicaBiosystems.com)

- Specimens, before and after fixation, and all materials exposed to them, should be handled as if capable of transmitting infection and disposed of with proper precautions<sup>2</sup>. Never pipette reagents by mouth and avoid contacting the skin and mucous membranes with reagents or specimens. If reagents or specimens come in contact with sensitive areas, wash with copious amounts of water. Seek medical advice.
- Consult Federal, State or local regulations for disposal of any potentially toxic components.
- Minimize microbial contamination of reagents or an increase in non-specific staining may occur.
- Retrieval, incubation times or temperatures other than those specified may give erroneous results. Any such change must be validated by the user.

### Instructions for Use

Calretinin (CAL6) primary antibody was developed for use on the automated BOND system (includes Leica BOND-MAX system and Leica BOND-III system) in combination with BOND Polymer Refine Detection. The recommended staining protocol for Calretinin (CAL6) primary antibody is IHC Protocol F. Heat induced epitope retrieval is recommended using BOND Epitope Retrieval Solution 2 for 20 minutes.

### Results Expected

#### Normal Tissues

Clone CAL6 detected the calretinin protein in some neurones in the CNS and peripheral nerve (especially large neurones), some thecal cells in ovary and Leydig cells and spermatogonia of testis. No additional staining other than nerve elements and occasional infiltrating cells was observed in a variety of other normal tissue evaluated. (Total number of normal cases evaluated =101).

#### Tumor Tissues

Clone CAL6 stained 3/4 mesotheliomas, 1/4 ovarian tumors and 2/3 skin tumors. No staining was observed in lung tumors (0/76), thyroid tumors (0/6), tumors of the colon (0/7), brain tumors (0/5), esophageal tumors (0/5), cervical tumors (0/5), kidney tumors (0/5), stomach tumors (0/5), breast tumors (0/5), tumors of the gallbladder (0/4), liver tumors (0/4), tumors of the pancreas (0/3), tumors of the bladder (0/3), testicular tumors (0/2), tumors of the prostate (0/2), metastatic tumors of unknown origin (0/2), tumors of the tongue (0/2), soft tissue tumors (0/2), a tumor of the larynx (0/1) and a tumor of the thymus (0/1). (Total number of abnormal cases evaluated = 156).

**Calretinin (CAL6) is recommended for the detection of human Calretinin protein in normal and neoplastic tissues, as an adjunct to conventional histopathology using non-immunologic histochemical stains.**

### Product Specific Limitations

Calretinin (CAL6) has been optimized at Leica Biosystems for use with BOND Polymer Refine Detection and BOND ancillary reagents. Users who deviate from recommended test procedures must accept responsibility for interpretation of patient results under these circumstances. The protocol times may vary, due to variation in tissue fixation and the effectiveness of antigen enhancement, and must be determined empirically. Negative reagent controls should be used when optimizing retrieval conditions and protocol times.

### Troubleshooting

Refer to reference 3 for remedial action.

Contact your local distributor or the regional office of Leica Biosystems to report unusual staining.

### Further Information

Further information on immunostaining with BOND reagents, under the headings Principle of the Procedure, Materials Required, Specimen Preparation, Quality Control, Assay Verification, Interpretation of Staining, Key to Symbols on Labels, and General Limitations can be found in "Using BOND Reagents" in your BOND user documentation.

### Bibliography

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Groves P, Palczewska M. Calcium binding properties of calretinin, an EF-hand calcium protein. Acta Biochemica Polonica 2001, 48(1), 113-119.
5. Rogers JH. Calretinin: a gene for a novel calcium-binding protein expressed principally in neurons. The Journal of Cell Biology 1987, 105, 1343-1353.

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## Bibliographie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code : M9-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Groves P, Palczewska M. Calcium binding properties of calretinin, an EF-hand calcium protein. Acta Biochemica Polonica 2001, 48(1), 113-119.
5. Rogers JH. Calretinin: a gene for a novel calcium-binding protein expressed principally in neurons. The Journal of Cell Biology 1987, 105, 1343-1353.

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## Bibliografie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 28. February 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD und Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Groves P, Palczewska M. Calcium binding properties of calretinin, an EF-hand calcium protein. Acta Biochemica Polonica 2001, 48(1), 113-119.
5. Rogers JH. Calretinin: a gene for a novel calcium-binding protein expressed principally in neurons. The Journal of Cell Biology 1987, 105, 1343-1353.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code : M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Groves P, Palczewska M. Calcium binding properties of calretinin, an EF-hand calcium protein. Acta Biochemica Polonica 2001, 48(1), 113-119.
5. Rogers JH. Calretinin: a gene for a novel calcium-binding protein expressed principally in neurons. The Journal of Cell Biology 1987, 105, 1343-1353.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Groves P, Palczewska M. Calcium binding properties of calretinin, an EF-hand calcium protein. Acta Biochemica Polonica 2001, 48(11), 113-119.
5. Rogers JH. Calretinin: a gene for a novel calcium-binding protein expressed principally in neurons. The Journal of Cell Biology 1987, 105, 1343-1353.

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Leica Biosystems Newcastle Ltd
Leica Biosystems Inc
Pty Ltd